

CCP8 FISH Probe

Introduction

Chromosome counting probe 8 (CCP8) FISH Probe is designed to detect the copy number of chromosome 8 or to serve as a control to determine the relative number of copies of genes located on chromosome 8 or other chromosomes. The probe is derived from chromosome 8 specific alpha satellite DNA.

Intended Use

To measure the copy number of the human chromosome 8.


Cont.

CCP8 FISH Probe

Color

CytoGreen

Probe Design

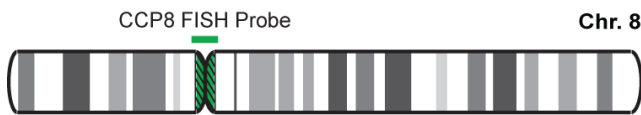

The CCP8 probe hybridizes to chromosome 8 in both metaphase and interphase cells. After hybridizing with normal human peripheral blood lymphocyte samples, two distinct bright fluorescent spots could be observed in the interphase nuclei under a fluorescence microscope. In metaphase cells, bright signals can be observed on the centromere region of chromosome 8 (8p11.1-q11.1). No cross-hybridization to loci on other chromosomes is observed.

Not to Scale

Cat. No.

CT-CCP008-10-G

Volume

10 Tests (100 µL)

Signal Pattern Interpretation

Normal Pattern

2G

Abnormal Pattern

Other Patterns

- 1) Jenkins RB, et al. *Blood*. 79(12):3307-15 (1992).
- 2) Escudier SM, et al. *Blood*. 81(10):2702-7 (1993).
- 3) Heim S & Mitelman F. *Cancer Cytogenetics 2nd Ed.* (1995).
- 4) Najfeld V, et al. *Bone Marrow Transplant*. 19(8):829-34 (1997).
- 5) Byrd JC, et al. *Clin Cancer Res*. 4(5):1235-41 (1998).



CytoTest Inc.
9430 Key West Ave., Suite 210
Rockville, MD 20850, USA

* CE IVD only available in certain countries. All other countries are either ASR or RUO. Please contact your local dealer or our headquarters for more information.

Life Technologies (India) Pvt Ltd.

306, Agarwal City Mall, Road 44, Pitampura, Delhi - 110034 (India)

Tel: +91-11-4220-8000; 4220-8111; 4220-8222 Fax: +91-11-4220-8444, Mobile: +91-98105-21400

Email - customerservice@lifetechindia.com | customerservice@atzlabs.com