

## MN1 Break Apart FISH Probe Kit

### Introduction

The MN1 Break Apart FISH Probe Kit is designed to detect rearrangements in the human *MN1* gene located on chromosome band 22q12.1. In addition to revealing breaks, which can lead to translocation of parts of the gene, inversion, or its fusion to other genes, the probe set can also be used to identify other *MN1* aberrations such as deletions or amplifications. Rearrangements and abnormal expression of the *MN1* gene – also known as *dJ353E16.2*, *MGCR1-PEN*, *MGCR1* or *MGCR* – have been observed in myelodysplastic syndrome (MDS), acute myeloid leukemia (AML), meningioma and other malignancies.

### Intended Use

To detect rearrangements in the human *MN1* gene located on chromosome band 22q12.1.


### Cont.

LSP MN1 5' FISH Probe  
LSP MN1 3' FISH Probe

### Color

CytoOrange  
CytoGreen

### Probe Design

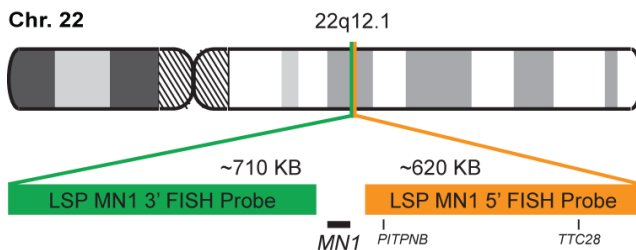

LSP MN1 5' FISH Probe covers sequences adjacent to the 5' (start) portion of the *MN1* gene. LSP MN1 3' FISH Probe covers some sequences downstream (3' end) of the gene. The two probes are flanking sequences across the *MN1* gene in which variable breakpoints have been observed.

Not to Scale

### Cat. No.

CT-PAC112-10-OG

### Volume

10 Tests (100 µL)

### Signal Pattern Interpretation

#### Normal Patterns

2F\*

#### Abnormal Patterns

Other Patterns

\*Overlapping orange and green signals can appear as yellow.

- 1) Buijs A, et al. *Oncogene*. 10(8):1511-9 (1995).
- 2) Lekanne Deprez RH, et al. *Oncogene*. 10(8):1521-8 (1995).
- 3) Heuser M, et al. *Blood*. 110(5):1639-47 (2007).
- 4) Nofrini V, et al. *Leuk Res*. 35(7):e123-6 (2011).
- 5) Aref S, et al. *Hematology*. 18(5):277-83 (2013).



CytoTest Inc.  
9430 Key West Ave., Suite 210  
Rockville, MD 20850, USA

\* CE IVD only available in certain countries. All other countries are either ASR or RUO. Please contact your local dealer or our headquarters for more information.

### Life Technologies (India) Pvt. Ltd.

306, Aggarwal City Mall, Road No. 44, Pitampura, Delhi – 110034, India

Mobile: +91-98105-21400, Tel: +91-11-42208000, 8111, 8222, Fax: +91-11-42208444

Email: [customerservice@lifetechindia.com](mailto:customerservice@lifetechindia.com), [www.atzlabs.com](http://www.atzlabs.com); [www.lifetechindia.com](http://www.lifetechindia.com)