

FGFR4 Break Apart FISH Probe Kit

Introduction

The FGFR4 Break Apart FISH Probe Kit is designed to detect rearrangements in the human *FGFR4* gene located on chromosome band 5q35.2. In addition to revealing breaks, which can lead to translocation of parts of the gene, inversion, or its fusion to other genes, the probe set can also be used to identify other *FGFR4* aberrations such as deletions or amplifications. Rearrangements and abnormal expression of the *FGFR4* gene – also known as *CD334*, *JTK2* or *TKF* – have been observed in various cancer types such as prostate cancer, melanoma, hepatocellular carcinoma, lung cancer, breast cancer, gastric cancer, colorectal cancer, pancreatic cancer and others.

Intended Use

To detect rearrangements in the human *FGFR4* gene located on chromosome band 5q35.2.


Cont.

LSP FGFR4 5' FISH Probe
LSP FGFR4 3' FISH Probe

Color

CytoOrange
CytoGreen

Probe Design

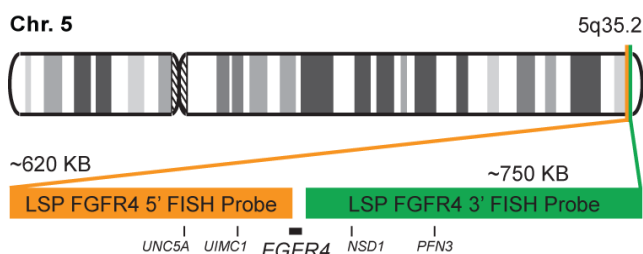

Not to Scale

LSP FGFR4 5' FISH Probe covers sequences adjacent to the 5' (start) portion of the *FGFR4* gene. LSP FGFR4 3' FISH Probe covers some sequences downstream (3' end) of the gene. The two probes are flanking sequences across the *FGFR4* gene in which variable breakpoints have been observed.

Cat. No.

CT-PAC133-10-OG

Volume

10 Tests (100 µL)

Signal Pattern Interpretation

Normal Patterns

2F*

Abnormal Patterns

Other Patterns

*Overlapping orange and green signals can appear as yellow.

- Partanen J, et al. *EMBO J.* 10(6):1347-54 (1991).
- Okada A, et al. *Proc Natl Acad Sci U S A.* 92(7):2730-4 (1995).
- Shah RN, et al. *Oncogene.* 21(54):8251-61 (2002).
- Ruhe JE, et al. *Cancer Res.* 67(23):11368-76 (2007).
- Xu W, et al. *Eur J Cancer.* 46(18):3332-8 (2010).



CytoTest Inc.
9430 Key West Ave., Suite 210
Rockville, MD 20850, USA

* CE IVD only available in certain countries. All other countries are either ASR or RUO. Please contact your local dealer or our headquarters for more information.

India Contact:

Life Technologies (India) Pvt. Ltd.

306, Aggarwal City Mall, Opposite M2K Pitampura, Delhi – 110034 (INDIA). Ph: +91-11-42208000, 42208111, 42208222, Mobile: +91-9810521400, Fax: +91-11-42208444
Email: customerservice@lifetechindia.com Website: www.lifetechindia.com