



Your Molecular & Cell Technology Partner

For Professional Use Only

CBFB Break Apart FISH Probe Kit

Introduction


The CBFB Break Apart FISH Probe Kit is designed to detect rearrangements in the human *CBFB* gene located on chromosome band 16q22.1. In addition to revealing breaks, which can lead to translocation of parts of the gene, inversion, or its fusion to other genes, the probe set can also be used to identify other *CBFB* aberrations such as deletions or amplifications. Rearrangements and abnormal expression of the *CBFB* gene – also known as *CBFb* or *PEBP2B* – have been observed in acute myeloid leukemia (AML) and other hematological malignancies.

Intended Use

To detect rearrangements in the human *CBFB* gene located on chromosome band 16q22.1.

Cont.	Color
LSP CBFB 5' FISH Probe	CytoOrange
LSP CBFB 3' FISH Probe	CytoGreen

Probe Design

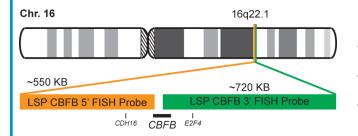

LSP CBFB 5' FISH Probe covers the 5' (start) portion of the *CBFB* gene and some adjacent genomic sequences. LSP CBFB 3' FISH Probe covers the 3' (end) part as well as sequences downstream of the gene. The two probes are flanking sequences across the *CBFB* gene in which variable breakpoints have been observed.

Not to Scale

Cat. No.	Volume
CT-PAC328-10-OG	10 Tests (100 μL)

Signal Pattern Interpretation

Normal Pattern

20G*

Abnormal Pattern

Other Patterns

*Overlapping orange and green signals can appear as yellow.

India Contact:

Life Technologies (India) Pvt. Ltd.

¹⁾ Liu P, et al. *Blood*. 82(3):716-21 (1993).

²⁾ Ogawa E, et al. Virology. 194(1):314-31 (1993).

³⁾ Ogawa E, et al. *Proc Natl Acad Sci U S A*. 90(14):6859-63 (1993).

⁴⁾ van der Reijden BA, et al. *Leukemia*. 10(2):204-6 (1996).

⁵⁾ Wang Q, et al. Cell. 87(4):697-708 (1996).

CytoTest Inc. 9430 Key West Ave., Suite 210 Rockville, MD 20850, USA

^{*} CE IVD only available in certain countries. All other countries are either ASR or RUO. Please contact your local dealer or our headquarters for more information.